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Direction of use

Mega- A

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1. Information

Mega-A

Cold curing resin for repairs and relinings

Shades : pink, blue-pink veined and clear

2. Composition

Powder/liquid cold cure denture resin

Main component powder: Methylmethacrylate, Copolymer

Component liquid: Methylmethacrylate

3. Indications

Dental denture repair resin for functional reconstruction, repair, partial and total re-lining of dentures

4. Contraindication

Allergic reactions in wearers to dental prosthesis base resin are rare. Placement of Mega-A is contraindicated if the patient is known to be allergic to any of the ingredients in Mega-F.

5. Side Effects

none admits

6. Warning

Mega-A liquid contains high flammable Methylmethacrylate. Irritating to eyes, respiratory system and skin. Avoid contact with skin, wear suitable gloves. Keep away from sources of ignition (no smoking) , keep out of reach of children. If swallowed, seek medical advice immediately and show the product label.

7. Application

- Preparations

Use hard plaster for model. Remove wax residue thoroughly from the resin parts. Insulate watered plaster model twice with a thin layer of MEGA-I insulation.

- Dosing and Handling

Recommended mixing ratio is 5 g powder : 3 ml liquid (20 s). The liquid is placed in a mixing vessel and the powder is added until a minor liquid level remains. The mixture is mixed intensively for 20 s. The pourable liquid dough can be used immediately. MEGA-A is applied evenly and have not to be touched or moulded after reaching the plastic consistency phase.

- Polymerization

Curing time at temperature of 21°C approximately 13 min. Optimum polymerization without air bubbles can be made in the pressure curing unit at 45 °C and 2 bars pressure for 10 min. If larger quantities are used and a pressure cure unit is not available, MEGA- A must be applied in layers. Each layer must cure individually and cooled down completely before new dough is applied.

8. Shelf life

Liquid: 2 years

Powder: 3 years

9. Precautions and Storage

Not above 25°C

10. Deliver units

Powder: 80 g / 500 g / 1000 g

Liquid : 80 ml / 500 ml / 1000 ml

11. Date information prepared

September 2010

12. Manufacture and sales

MEGADENTA Dentalprodukte GmbH

D-01454 Radeberg, Germany